

Surgical Instrument Detergent Enzyme Cleaner

Then all-in-ONE is highly concentrated for fast and cost effective: presoaking, hand washing, ultrasonic cleaning, and automated washing.

The all-in-ONE Enzyme Detergent Cleaner is: non-irritating, 100% biodegradable, phosphate free, neutral pH,

- breaks down all forms of bioburden and proteinaceous debris,
- cleans and conditions the surface,
- removes encrustation and stains,
- builds protection against pitting and corrosion,
- lubricates moving parts, and then
- rinses clean, for a residue free surface.

The all-in-ONE delivers the 4 Enzymes that are necessary to break down all forms of bioburden (Protease, Lipase, Amylase, and Carbohydrase) and then rinses clean. Enzyme Detergent Cleaners that do not provide these 4 enzymes cannot remove all forms of bioburden and proteinaceous debris. The high concentration level will cut costs while delivering the most effective cleaning results. 1 gallon (3.78 liters) delivers 512 gallons (1938 liters) of cleaning power. The dilution rate is .25 to .5 ounces per gallon of diluent (1.9 ml liters to 3.8 ml per liter of diluent) depending on the level of bioburden, encrustation, and stains. When used as directed, the all-in-ONE Surgical Instrument Detergent Enzyme Cleaner replaces multiple products, lowers reprocessing costs, improves the performance of surgical instruments, renders the highest quality cleaning outcomes, and is non-irritating to the nose, eyes, throat, and skin.

The easy FOAM-it, all-in-ONE pre wash cleaner, lays down a thick blanket of long lasting FOAM to hydrate and remove all forms of proteinaceous debris, hard water deposits, and encrusted stains, for fast cleaning action. When water is added the FOAM dissipates, providing a clear 'free-rinsing' surgical instrument detergent enzyme cleaning and conditioning solution.

The all-in-ONE Enzyme Detergent comes pre-loaded into Surgical Instrument and Endoscope cleaning Sponges offering an additional Margin of Safety with high quality cleaning outcomes.



all-in-ONE easy FOAM-it,



all-in-ONE Enzyme Detergent Cleaners.

The highly concentrated all-in-ONE cleaners are aggressive against your most difficult cleaning challenge and guaranteed to be safe for all types of metals, rigid and flexible scopes, lenses, microsurgical instrumentation, and automated reprocessing equipment.

- ➔ significantly reduce the cost of enzymatic cleaners, detergents, stain removers, and lubricants,
- ➔ decrease inventory levels, storage space and freight costs,
- ➔ improve the overall performance and appearance of surgical instruments and scopes,
- ➔ combine the enzyme, detergent, conditioning, and lubrication treatments for automated washers, manual cleaning, and presoaking,
- ➔ improve turnaround times for soaking, hand washing, and automated cleaning,

UPS-FS-32 easy FOAM-it 32 ounce FOAM applicators: 12 per case

Ergo UPS1-1 all-in-ONE 4 gallons with 1 pump per case

Ergo UPS1-5 all-in-ONE: 1 ea. 5 gallon jug

Ergo UPS1-15 all-in-ONE: 1 ea. 15 gallon drum

Ergo UPS1-30 all-in-ONE: 1 ea. 30 gallon drum

Ergo UPS1-55 all-in-ONE: 1 ea. 55 gallon drum

The TOSI Test provides an indication of cleaning performance. The amount of red material (image to the right)

simulates proteinaceous bioburden remaining after soaking for 4 minutes at 135 degrees F.

The Test Strip on the Left shows cleaning results using the RUHOF Endozime AW Triple Plus.

The Test Strip on the Right shows cleaning results using the all-in-ONE Detergent 4 Enzyme Cleaner.

Your greatest cleaning challenge will be your best proof of all-in-ONE cleaning performance.

Nothing cleans faster or more effectively than the all-in-ONE Detergent 4 Enzyme Cleaner.

Nothing lowers cleaning costs as much as the all-in-ONE Detergent 4 Enzyme Cleaner, we guarantee it.



Customer Service and Product Support: CustomerService@ergo-Logistics.com 1-509-747-5027

Technical Support: Support@ergoLogistics.com 1-509-981-1553

Websites for additional information: www.ergo-Logistics.com www.Surgical-Instrument-Care.com